

## OPERATING MANUAL



Chair for taking blood samples

CE certified according to "Direktivet för Medicintekniska Produkter 93/42/EEG inklusive tillägg".

<b>Frame:</b>	Five footed base.
<b>Seat:</b>	Rotates, stepless adjustable sitting angle, height 43-56 cm. Vertically adjustable by means of a gas spring.
<b>Back support:</b>	Adjustable height and angle, angle adjustable by means of a gasspring.
<b>Arm support:</b>	Stable, concave armrests, fully and easily adjusted. The armrest pad can be turned and used on both sides..
<b>Upholstery:</b>	Imitation leather.
<b>Patient handgrips:</b>	Adjustable in width between 60-70 cm.

Postal address

Myrångsvägen 8

SE-891 50 Örnsköldsvik, SWEDEN

Telephone

+46 (0)660-105 80

Fax

+46 (0)660-180 00

Web-site

[www.tarsus.se](http://www.tarsus.se)

E-mail

[info@tarsus.se](mailto:info@tarsus.se)




<u>Pos</u>	<u>Item No.</u>	<u>Designation</u>
1	010-5031	Pads for arm rests
2	010-5032	Arm rests
3	010-5033	Knob for lateral adjustment of patient handgrips
4	-	Adjustment for the backrest
5	010-5035	Knob for vertical adjustments
6	-	Adjustment of the angle of the backrests and seat angle
7	-	Height adjustments by means of a gas spring
8	010-5039	Patient handgrips
9	010-5040	Gas spring


### Maintenance and cleaning

Pads on armrests are turnable and easy to take off for cleaning. Clean the chair with warm water and a soap solution. For disinfection, use Isopropanol PEG40, Hydrogenated Castor Oil Water or equivalent.

### Assembly

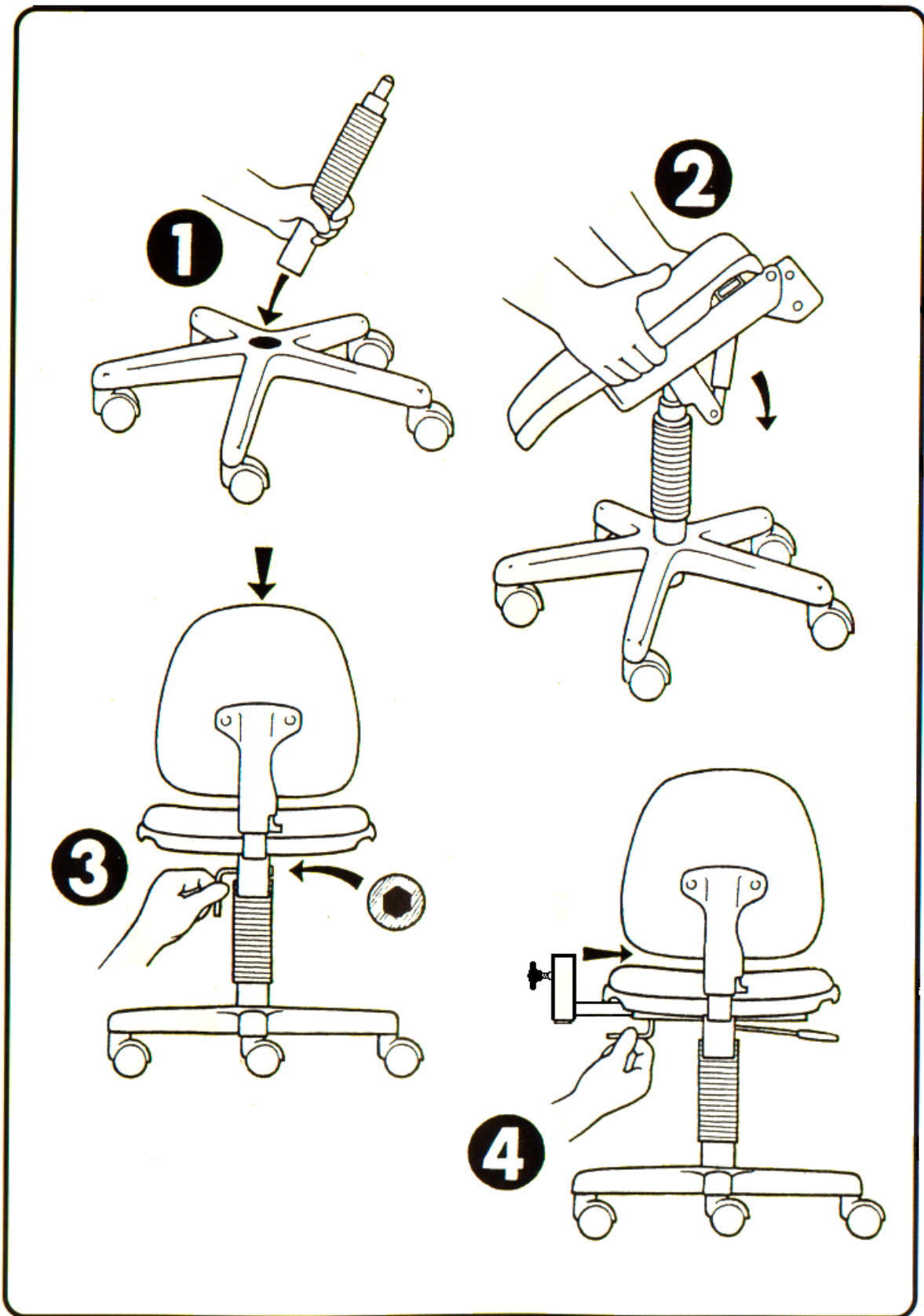

Mounting the patient handgrips

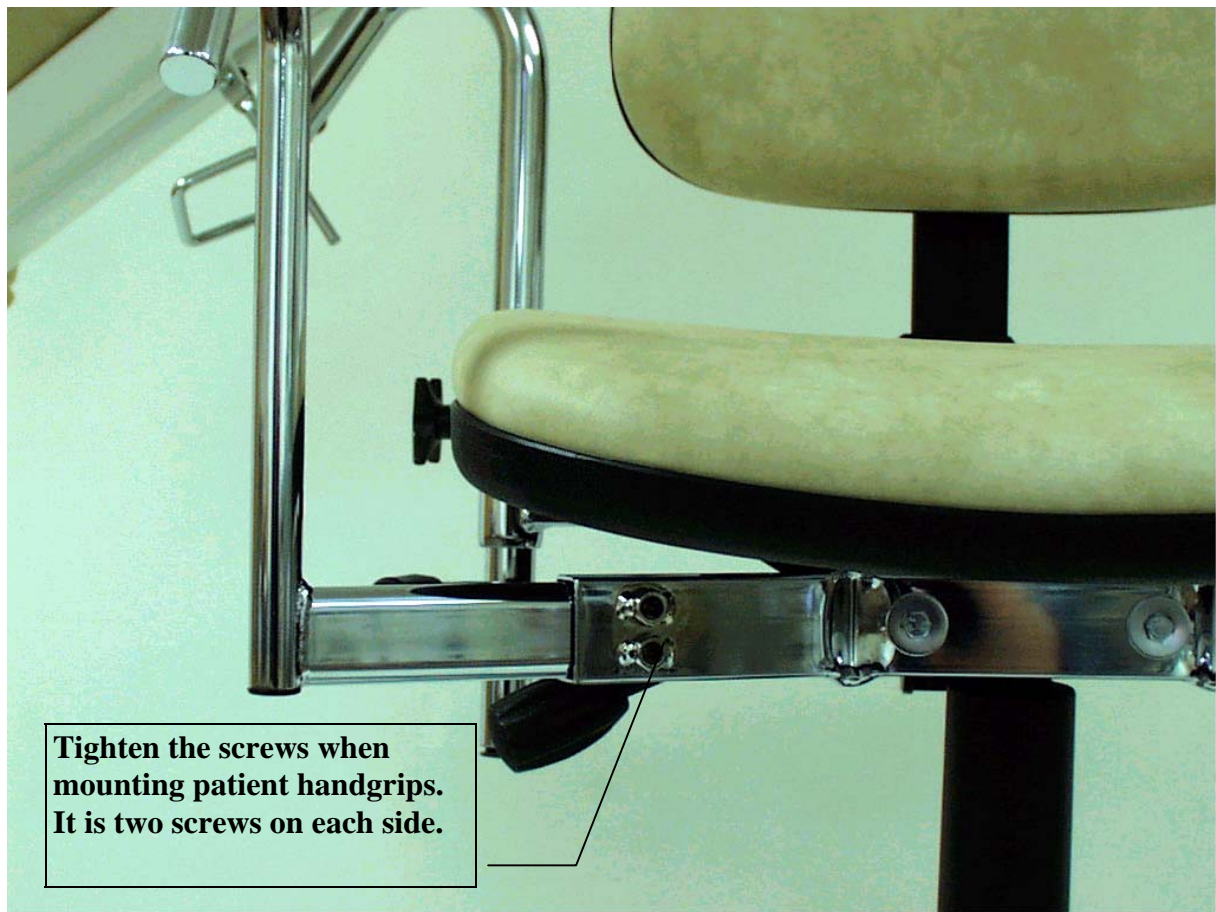

## **TARSUS PRODUCTS AB**

**Postal address**

Myrängsvägen 8

SE-891 95 Örnsköldsvik

SWEDEN

**Telephone**

+46 (0) 660-105 80

**Fax**

+46 (0) 660-180 00

**Web-site**

[www.tarsus.se](http://www.tarsus.se)

**E-mail**

[info@tarsus.se](mailto:info@tarsus.se)