

tarsus

OWNERS MANUAL



Infusions stand

- 080-3050 Without handle
- 080-3110 With handle
- 080-3200 For dosage pump, with handle
- 080-3210 For dosage pump, without handle

NOTE! To avoid unnecessary risks for both the patient and the operator, the operator MUST read through the Operation and Service Manual thoroughly before using the product.

Tarsus Products
Myrångsvägen 8
891 50 Örnsköldsvik, SWEDEN

Phone +46 (0)660 105 80
Webb www.tarsus.se

Fax +46 (0)660 180 00
E-mail info@tarsus.se

Contents

Foreword

Warranty

Important information

Specification of accessories

How to mount

Handle

Maintenance and cleaning

Description, technical documentation

Foreword

Welcome as owner of a Tarsus Infusion stand produced by Sjöbloms Sjukvårdsutrustning AB of Sweden. We hope that you will be pleased with this quality product. Your satisfaction is our goal. If a problem arises, you are of course always welcome to contact our service department, and our expert staff will assist you to solve the problem in every way needed.

This Operation and Service Manual contains all needed information for usage, maintenance and service of the product, as well as instructions on how to exchange spare parts. The Operation and Service Manual is being up-dated on a regular basis when needed, and when the product is being modified. Future changes in the manual may therefore occur, for which we make a certain reservation.

Warranty

All Tarsus products are warranted for two (2) years against defeats and in workmanship and materials. The warranty is void if the product is improperly used or altered. The warranty does not include normal wear and tear. The warranty does not include damage or defect caused by accident, misuse or abnormal working conditions. It is important that needed service and maintenance is executed according to instructions in this manual.

Important information

The Operation and Service Manual must be read thoroughly before using the product, to avoid unnecessary risks for both patient and operator.



The Operation and Service Manual contains warnings, noting where risks for personal injury may occur. These warnings must be noticed and respected.

A warning triangle is shown where very important information could be found.



Warning triangle

Specification of accessories

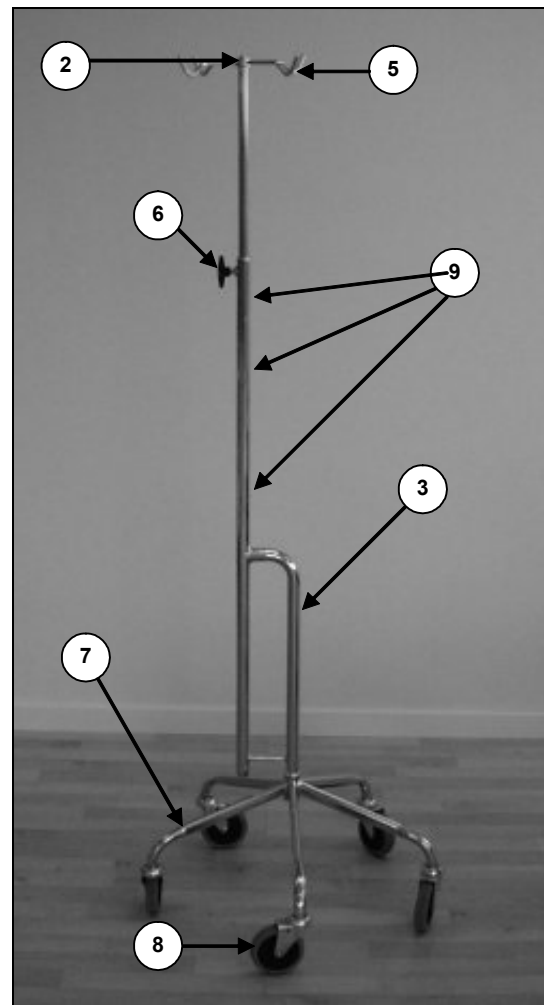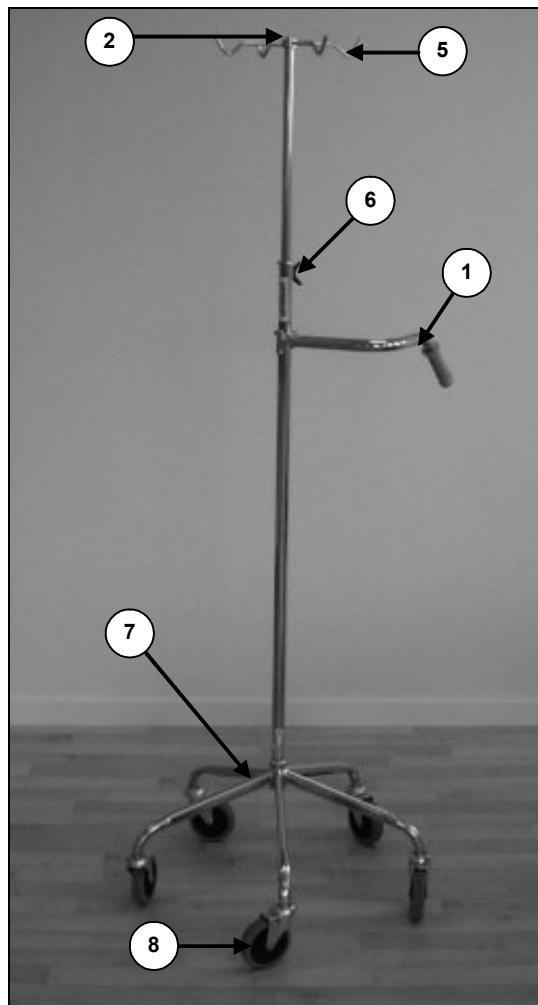

<u>Pos</u>	<u>Art no</u>	<u>Designation</u>
1	080-3006	4-pointed Hook
(Is being delivered, taped on under the five-legged castor base)		
2	080-3007	Locking knob for adjustment of height
3	080-3008	Five-legged castor base
4	080-3009	Castors, 100 mm
5	080-3002	Steering handle
6	080-3003	Screw for the 4-pointed hook
7	080-3004	Pump holder
8	080-3005	Bolt and rounded washer for mounting of the outer tube
	080-3001	Antistatic Castors


NOTE!

Maximum loading capacity for the dosage pump model: 3 pumps. Place the pumps so that they are mounted at the centre of the pump (see nr 9).


How to mount

If the infusion stand is delivered with the four-hook not mounted, do as follows: Unwind the screw at the top of the inner tube (Pos 2 on the Specification list). Fastened the hook to the inner tube. Make sure it is properly fastened.


Handle

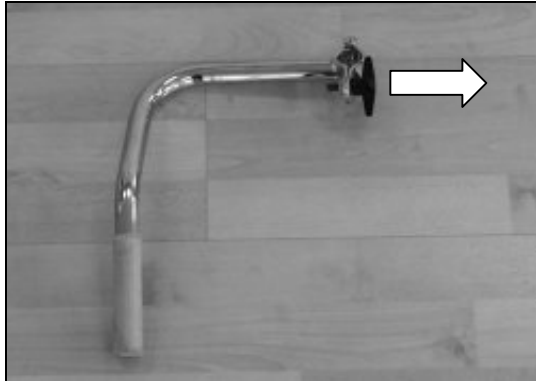

Unwind the locking screw.

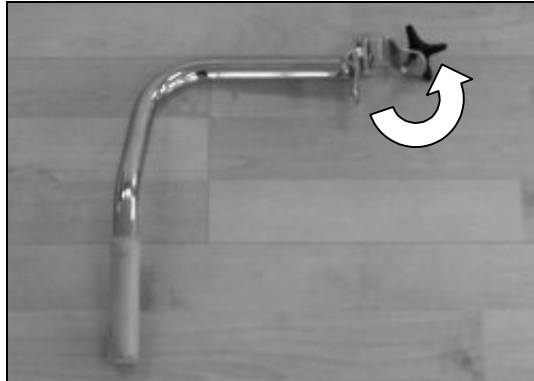

Fold out.

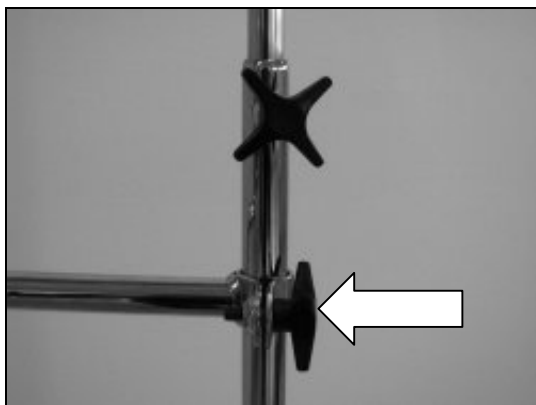

Connect the handle to the infusion stand and lock properly.



If the infusion stand is delivered with the outer tube not mounted, do as follows: Place the tube into the foot (images 1-2) and tighten properly (image 3).



NOTE! Tighten properly.



Maintenance and cleaning

The infusion stand can be cleansed by wiping off with spirit, (rubber castors and plastic details **NOT** included).

The infusion stand is basically maintenance free but routine controls must be done to ensure all functions.



Omission of the following routine check-ups may expose the patient to unnecessary risks and may reduce the life length of the table. Therefore, check all the functions and parts of the stand on a regular basis.

Control following:

- Noise from the castors
- That the castors swivels freely and without problems
- The condition of the locking screw, and that the tube is properly fastened.
- The locking bolts, that they are fastened properly.

Description, technical documentation

Designs

The stand is equipped with four hooks and a five-castor base. It's available with or without steering handle. The steering handle is detachable. NOTE! The expander construction of the knob locking provides a smooth and efficient locking when raising and lowering the tube with the 4-pointed hook.

Stand

All-welded, chrome plated.

Castors

100 mm. Antistatic castors are available.

Measure & weight

Measure

Minimum height: 130 cm. Maximum height: 220 cm.

Weight

5,5 kg.